




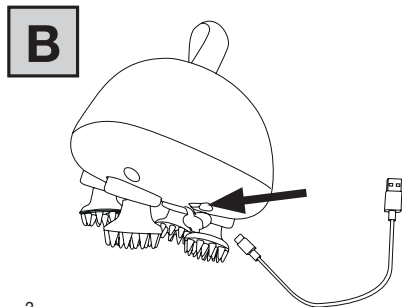

EN	Massager	
	Instructions for use.....	4

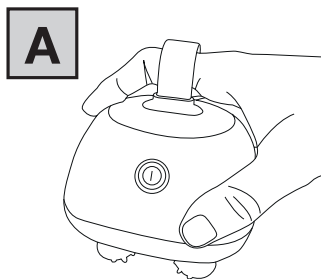

ENGLISH

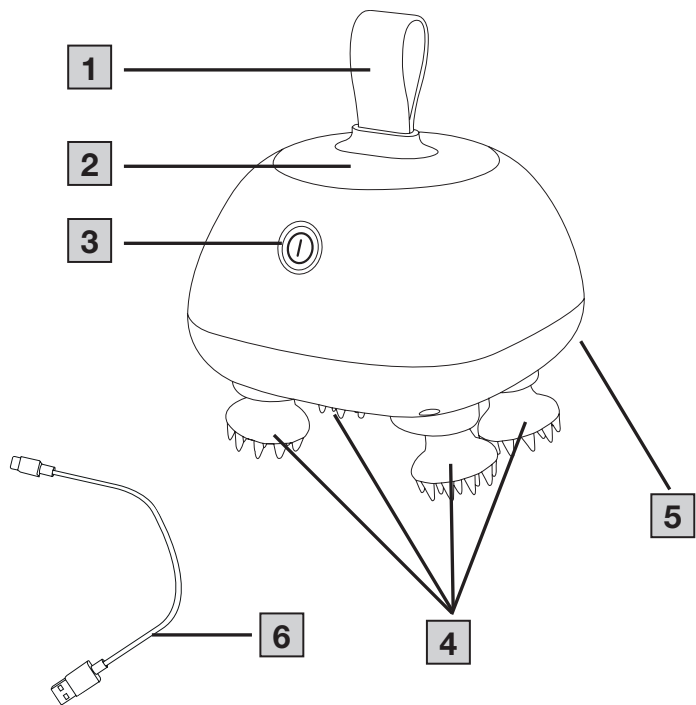

Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the next user with these instructions for use.

CONTENTS

1. Included in delivery	4	7. Cleaning and maintenance.....	8
2. Signs and symbols	4	8. What if there are problems?	9
3. Intended Purpose	5	9. Disposal.....	9
4. Warnings and safety notes.....	5	10. Technical specifications	10
5. Device description.....	7	11. Guarantee.....	10
6. Usage	7		
6.1 Initial use.....	7		
6.2 Starting use.....	8		

1. INCLUDED IN DELIVERY

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device and accessories and/or replacement parts, and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 massager
- 4 removable massage attachments
- 1 USB charging cable

2. SIGNS AND SYMBOLS

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

WARNING

Indicates a potentially imminent hazard. If it is not avoided, there is a risk of death or serious injury.

CAUTION

Indicates a potentially imminent hazard. If it is not avoided, slight or minor injuries may result.

NOTICE

Indicates a potentially harmful situation. If it is not avoided, the device or something in its vicinity may be damaged.



Product information

Note on important information



Read the instructions












Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).



Do not dispose of batteries containing hazardous substances with household waste



Manufacturer

	CE labelling This product satisfies the requirements of the applicable European and national directives.
	Separate the product and packaging elements and dispose of them in accordance with local regulations.
	Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.
	Direct current
	USB-C connection
	United Kingdom Conformity Assessed Mark
	Importer
	Protection class III

3. INTENDED PURPOSE

This device is only intended for massaging individual body parts of dogs and cats. The device is only intended for domestic/private use. The device is NOT intended for medical or commercial use.

Do not use the device if one or several of the following warnings applies to your dog or cat. If you are unsure whether the device is suitable for your dog or cat, please consult your veterinarian.

The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

4. WARNINGS AND SAFETY NOTES

WARNING

- Danger of suffocation by packaging material! Keep children and your dog/cat away from the packaging material.
- Inspect the device and accessories for visible damage before use. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- Do not use the device on humans.
- If your dog/cat is injured, consult your veterinarian before using the device.
- Do not use the device if you or your dog/cat suffer from skin problems.
- Do not use the device on swollen, burnt, inflamed or injured skin and body parts.
- Do not use the device on the face (eyes), the larynx or other particularly sensitive body parts.
- Do NOT use the device on the area around the heart.
- Only use the device together with the massage attachments.
- Only use the device on the soft surfaces of your dog/cat. Do not massage one area for longer than 60 seconds at a time.
- Danger of crushing! Keep your fingers away from the moving massage attachments during use.
- Keep your hair away from the moving massage attachments during use to avoid getting tangled.
- When using the device in the bathroom, unplug the mains plug after use as water in the vicinity constitutes a danger, even when the device is switched off.
- Keep the device away from extended exposure to direct sunlight, extreme heat or boiling water.
- Avoid using corrosive cleaning products or care products which are clay or silicone based.

- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.

Repairs

WARNING

Only specialist personnel may perform repairs on electrical devices. Improper repairs may put users in significant danger. For repairs, please contact Customer Services or an authorised retailer.

Risk of fire!

WARNING

There may be a risk of fire if the device is used improperly or these instructions for use are ignored. For that reason, never operate the device unsupervised, especially when children are nearby and never operate it under a cover such as a blanket or pillow.

Handling

CAUTION

- The device must be switched off after every use and before cleaning.
- Ensure that hair does not get trapped in the device.
- Avoid contact with pointed or sharp objects.
- Protect the device from sources of heat getting too close to the device, such as ovens, heaters or direct sunlight.

Risk of electric shock

WARNING

Make sure that the device and the cable do not come into contact with water, steam or other liquids. For that reason, only use the device in dry indoor areas (e.g. never in the bath, sauna). Do not use the device if the device or accessories show visible damage. Do not drop the device or subject it to any impacts.

CAUTION

Only specialist personnel may perform repairs on electrical devices. Improper repairs may put users in significant danger. In the event of a fault or damage, have the device repaired by a certified specialist workshop.

NOTICE

Notes on handling batteries

WARNING

- **Risk of explosion! Risk of fire!** Failure to comply with the points mentioned can result in personal injury, overheating, leakage, venting, breakage, explosion, or fire.
- Always use the correct or supplied charging cable for charging.
- Avoid continuous charging or overcharging. Unplug the charger when charging is complete.
- Only use with the appropriate mains adapter.
- Charge the device under supervision, paying attention to any heat generated, deformation or release of gases. If in doubt, stop charging.
- Dispose of defective batteries/charging cables/chargers immediately and properly (see chapter on disposal).
- Do not throw the device or batteries into a fire.
- Never forcibly discharge, heat, disassemble, open, crush, deform, encapsulate, modify or knock the device or batteries.

- Never short-circuit batteries or the connections of the battery-powered device.
- Protect the device or batteries from direct sunlight, rain, heat and water.
- Exposure of batteries to an environment with extremely high temperatures or an extremely low air pressure may result in explosion or leakage of flammable liquids and gases.
- If fluid from a battery comes into contact with your skin or eyes, wash the affected areas with water and seek medical assistance.
- Do not use modified or damaged batteries/charging cables/chargers.
- Always select the correct battery type. Contact Customer Services for information on whether a battery replacement part can be replaced.
- Always insert the batteries correctly, taking into account the polarity (+ / -).
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- **Choking hazard!** Keep batteries out of the reach of children and your dog/cat. Seek medical attention immediately if swallowed. Swallowing them may cause severe internal burns and death.
- Never allow children to replace batteries without adult supervision.

⚠ CAUTION

- Store battery replacement parts away from metal objects in a well-ventilated, dry, and cool place.
- Keep batteries clean and dry.
- Keep batteries away from water.
- If the device is not going to be used for a long period of time, remove the batteries from the battery compartment.

NOTICE

- This device contains a battery that is not replaceable. Once a battery has reached the end of its service life, the device must

be disposed of properly (see chapter on disposal).

5. DEVICE DESCRIPTION

The associated drawings are shown on page 3.

- 1 Finger loop
- 2 Battery cover
- 3 ON/OFF button
- 4 Massage attachments
- 5 USB-C connection
- 6 USB-C charging cable

6. USAGE

6.1 Initial use

You must fully charge the device for approx. 100 minutes before the first use. The operating period of the device is approx. 165 minutes when it is fully charged.

To do this, proceed as follows:

1. Connect the USB-C charging cable **6** to the USB-C connection socket **5** and a suitable mains adapter (not included in delivery).
2. Insert the mains adapter into a suitable socket. The ON/OFF button **3** lights up white during charging (see illustrations **B** on folding page 3).

NOTICE

The device cannot be used while it is being charged.

3. As soon as the ON/OFF button lights up **3** white, the battery is fully charged. When you remove the USB charging cable from the device, the ON/OFF button goes out **3**. The device is now ready for use.



When the battery life is less than 20%, the ON/OFF button starts [3] to flash. In "low speed" mode it starts flashing white – in "high speed" mode it starts flashing blue. The device has an automatic switch-off function and switches off automatically after 15 minutes of operation.

6.2 Starting use

1. To switch the device on, press and hold the ON/OFF button [3] for 3 seconds.
 - ↳ The ON/OFF button [3] starts to light up white. The massage attachments will start to rotate.
2. The device features a slow, medium and fast speed setting. You can switch between the two speed settings by briefly pressing the ON/OFF button [3].
 - ↳ The colour of the ON/OFF button [3] indicates the current speed setting:
 - white = low speed
 - blue = high speed
3. Slowly bring your dog/cat close to the device so that it can get used to the device and relieve any fears or reservations.
4. After the acclimation period, guide the device over the area to be treated on your dog/cat. Only use the device on the soft surfaces of your dog/cat. Do not massage one area for longer than 60 seconds at a time.

⚠ CAUTION

If the dog/cat feels uncomfortable or shows any signs of discomfort, stop using the device immediately.

5. Switch off the device after every use. To switch the device off, press and hold the ON/OFF button [3] for 2 seconds.
 - ↳ The ON/OFF button [3] goes out. The massage attachments will stop rotating.
6. Store the device in its original packaging after each use. Do not leave the device lying around.

⚠ WARNING

Always keep the device out of reach of children and pets.

7. CLEANING AND MAINTENANCE

Cleaning massage attachments

NOTICE

Only clean the massage attachments [4] by hand. Do NOT clean the massage attachments in the dishwasher or washing machine.

Clean the massage attachments [4] after each use as follows:

1. Unscrew the four massage attachments [4] from the device by hand.
2. Clean the massage attachments under running water.
3. Then allow the massage attachments to dry completely.
4. Screw the four fully dried massage attachments back onto the device.

Cleaning the device

- Never clean the device with cleaning products containing alcohol, petrol or acetone. Clean the device using only the methods specified.
- The device can be cleaned with a soft, slightly damp cloth. Do not use the device again until it is completely dry.

⚠ CAUTION

Ensure that no water enters the device. If this should happen, do not use the device again until it has fully dried out.

Storage

Always store the device in the original packaging after use and do not put anything heavy on it. Always keep the device out of reach of children and pets.

8. WHAT IF THERE ARE PROBLEMS?

Problem	Solution
The device will not switch on	When switching the device on, ensure that you press the ON/OFF button with sufficient force and hold the button down for a sufficient amount of time. Charge the device as described in these instructions for use. If it is still not possible to switch on the device, contact Customer Services.

9. DISPOSAL

For environmental reasons, do not dispose of the device in household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Observe the local regulations for material disposal. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal. You can obtain the location of collection points for old devices from the local authorities.



Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = battery contains lead,

Cd = battery contains cadmium

Hg = battery contains mercury




Environmentally friendly disposal

For the sake of the environment, we recommend you to dispose of the individual components separately, especially the batteries.

1.


To open the battery compartment lid

2., grasp the battery compartment lid with your hands and turn it clockwise.

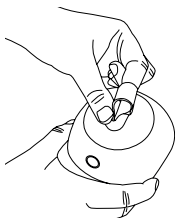


2.

After opening the battery compartment lid, you should see a red/black cable and a (white) plug connection that controls the power supply to the device via the rechargeable battery/battery.

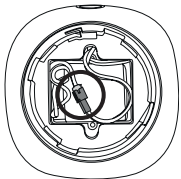


3.

Disconnect the plug connections. If you cannot see the white plug connection, it may have slipped into the housing of your device during operation.

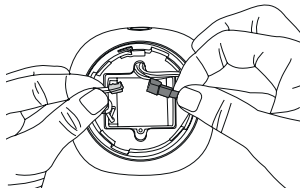

4.

Remove the battery from the device. Dispose of all components properly.

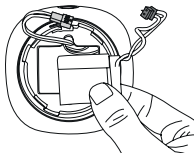

10. TECHNICAL SPECIFICATIONS

Device input	5 V $\overline{\text{---}}$ 1 A
Dimensions	approx. 102 x 102 x 98 mm (without loop)
Weight	approx. 302 g (without USB-C cable)
<u>Battery:</u> Capacity Nominal voltage Type designation	1200 mAh 3.7 V Lithium-ion battery

11. GUARANTEE

Further information on the guarantee and guarantee conditions can be found in the guarantee leaflet supplied.



Beurer GmbH • Söflinger Str. 218 • 89077 Ulm, Germany
www.beurer.com • www.beurer-gesundheitsratgeber.com
www.beurer-healthguide.com



105.24_PP410_2024-11-26_01_IM2_BEU_EN